Aquasonic® 100 Ultrasound Transmission Gel Single-Use Packettes

REF 01-20

- EN** Instructions for use
- ES** Instrucciones de uso
- DE** Gebrauchsanweisung
- FR** Mode d'emploi
- IT** Istruzioni per l'uso
- PT** Instruções de utilização
- DA** Brugsanvisning
- FI** Käyttöohjeet
- NL** Gebruiksaanwijzing
- SV** Bruksanvisning
- EL** Οδηγίες Χρήσης
- HU** Használati útmutatás
- PO** Instrukcja stosowania
- HE** הוראות שימוש
- JA** 取扱説明書
- KO** 사용법
- ZH** 使用说明
- AR** إرشادات الاستعمال

Parker Laboratories, Inc.
286 Eldridge Road
Fairfield, NJ 07004 USA
(973) 276-9500
www.parkerlabs.com

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Instructions for Use:

Description

Aquasonic 100 Ultrasound Transmission Gel Single-Use Packettes are non-sterile, water-soluble and non-staining gel intended for external use and natural body orifice* only, and may be used with all commercially available ultrasound devices. For single use only.

Indications for Use

Aquasonic 100 Ultrasound Transmission Gel Single-Use Packettes can be used on intact, unbroken skin and natural body orifice* and on all patients in facilities where cross contamination is of concern, such as:

- Patients with immunodeficiency or on immunosuppressive therapy.
- Neonates or critically ill pediatric patients.
- Mucous membranes*.
- Critically ill patients or patients in contact, airborne or droplet transmission based precautions.

Discard packette after use.

**Not to be used for procedures in the esophagus.*

Contraindications

Aquasonic 100 Ultrasound Transmission Gel Single-Use Packettes should not be used in:

- Any invasive procedures in which a device is passed through the tissue, e.g. needle aspiration, needle localization, tissue biopsy, or other surgical procedures.
- Any procedure on or near broken skin, such as fresh surgical site or an open wound.

Storage Conditions

Store in a cool dry place. Store away from direct sunlight.

Warnings and Precautions

Discard product if package is damaged or evidence of contamination is present. Do not use on patients with a known sensitivity to parabens.

Apply Aquasonic 100 Ultrasound Transmission Gel to transducer surface, and not directly on to the patient.

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