# AQUAFLEX® Ultrasound Gel Pad

REF 04-02

- EN** Instructions for use
- ES** Instrucciones de uso
- DE** Gebrauchsanweisung
- FR** Mode d'emploi
- IT** Istruzioni per l'uso
- PT** Instruções de utilização
- DA** Brugsanvisning
- FI** Käyttöohjeet
- NL** Gebruiksaanwijzing
- SV** Bruksanvisning
- EL** Οδηγίες Χρήσης
- HU** Használati útmutatás
- PO** Instrukcja stosowania
- HE** הוראות שימוש
- JA** 取扱説明書
- KO** 사용법
- ZH** 使用说明
- AR** إرشادات الاستعمال

Parker Laboratories, Inc.  
286 Eldridge Road  
Fairfield, NJ 07004 USA  
(973) 276-9500  
www.parkerlabs.com

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## Instructions for Use:

### Description

AQUAFLEX is an aqueous, flexible, disposable ultrasound standoff. AQUAFLEX cushions sensitive areas for comfortable scanning and/or therapeutic treatment of irregular surfaces such as bony prominences and small joints. The AQUAFLEX Ultrasound Gel Pad is solid, yet contours to irregular surfaces for accurate transmission.

### Indications for Use

AQUAFLEX is recommended for diagnostic procedures performed on:

- Anterior gall bladder
- Breasts
- Carotid artery
- Neonatal heads
- Subcutaneous masses
- Testicles
- Thyroid
- Any procedure where near field visualization is required

### Warnings

- Do not use on patients with a known history of skin allergies to parabens.

### Precautions

- For external use only on intact, unbroken skin.
- Discard product if package is damaged or evidence of contamination is present.

### Directions for Use

Refer to the Instructions For Use provided by the manufacturer of the ultrasound device being used for information on how AQUAFLEX should be used with their product.

### Handling

No special handling required.

### Storage

Protect product from direct sunlight.

### Disposal

Follow all federal, state, and local regulations for non-hazardous waste disposal.

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