Signagel® Electrode Gel

REF 15-25, 15-60

- EN** Instructions for use
- ES** Instrucciones de uso
- DE** Gebrauchsanweisung
- FR** Mode d'emploi
- IT** Istruzioni per l'uso
- PT** Instruções de utilização
- DA** Brugsanvisning
- FI** Käyttöohjeet
- NL** Gebruiksaanwijzing
- SV** Bruksanvisning
- EL** Οδηγίες Χρήσης
- HU** Használati útmutatás
- PO** Instrukcja stosowania
- HE** הוראות שימוש
- JA** 取扱説明書
- KO** 사용법
- ZH** 使用说明
- AR** إرشادات الاستعمال

PARKER LABORATORIES, INC.

Parker Laboratories, Inc.
286 Eldridge Road
Fairfield, NJ 07004 USA
(973) 276-9500
parkerlabs.com

Parker Laboratories BV
Bedrijvenpark Twente 305
7602 KL Almelo
The Netherlands

EN

Signagel® Electrode Gel

Instructions for Use:

Description

Signagel Electrode Gel is a non-sterile, multipurpose, electrically conductive, water-soluble, non-staining, non-gritty electrode gel. It is bacteriostatic.

Indications for Use

Signagel is recommended for all electromedical procedures including ECG, defibrillation, biofeedback and EMG.

Warnings

- Do not use on patients with a known history of skin allergies to parabens.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the EU Member State in which the user and/or patient is established.

Precautions

- For external use only on intact, unbroken skin.
- Discard product if package is damaged or evidence of contamination is present.
- To avoid contamination of product, the tube container tips should not come into contact with patient, staff or instruments.

Directions for Use

Refer to the Instructions For Use provided by the manufacturer of the electromedical device being used for information on how Signagel should be used with their product. Signagel is water soluble, and can be removed using a moistened soft cloth.

Handling

No special handling required. Signagel is non-flammable and non-corrosive.

Storage

Protect product from direct sunlight.

Disposal

Follow all federal, state, and local regulations for non-hazardous waste disposal.

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El gel para electrodos Signagel es un gel para electrodos no arenoso, antimanchas, electricamente conductor, polivalente y sin esterilizar. Es bacteriostático.

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