

# ULTRA DRAPE™

UGPIV Barrier and Securement

## Application Instructions

STERILE R



Assess location of the target vein using ultrasound, and most importantly, **mark the site**. Clean and disinfect skin to your facility's protocol. Before applying UltraDrape, position the dressing with blue tab towards you and pull away white paper. ① ↓



Bring the non-adherent sides together by folding dressing and holding at the tabs. Center the folded dressing at your **marked** target vein, then press and adhere the dressing's rear half (the portion without the split) on the skin. The lower half with the split will stand upright and away from the skin and the target insertion site.



Apply gel to the rear half (adhered dressing portion) and **increase gain** on the ultrasound unit. Use ultrasound to reacquire the target vein, positioning the probe while lightly holding dressing with thumb. Adjust ultrasound image and gain to your brightness preference. Inject lidocaine (Optional).



Using ultrasound guidance, insert the IV catheter into target vein, verify blood return, and advance the catheter as per policy and instructions. Complete the insertion and flush to confirm patency in the vein.



Carefully grasp the rear blue tab. ② ↓ Pulling from the side, remove gel layer and discard.



While holding two fingers on the adherent side of the dressing, grasp the blue tab ③ ↑ at the top of the split portion. Pull the tab up and off, and once removed, set aside the release film with the supplied adhesive tape strips.



Press dressing down over IV and tubing and bring tubing through the split section. Use the tape strips as needed to close the split.



Secure catheter and tubing with tape and document date on the dressing.

