



After scouting with ultrasound, mark the skin to target vessel. Prepare skin to your facility's protocol. Hold UltraDrape dressing with blue tab towards you and remove white paper. ① ↓



Center the dressing above your target vessel, then place and adhere the dressing's rear half (the portion without the split) on the skin. The lower half with the split will stand upright and away from the skin and the target insertion site.



Apply ultrasound gel on the adhered dressing portion and use ultrasound to reacquire the target vessel.



Use ultrasound guidance to insert the IV needle.



Once the placement of the IV catheter is confirmed, carefully grasp the blue tab ② ↓ and remove the top layer film cover where the gel was placed, discard.



Carefully grasping the blue tab ③ ↑ at the top of the split portion of the dressing, remove the film cover by pulling away from the insertion site while stabilizing the adherent side of the dressing.



Press the dressing over the IV and tubing, using the split section to cross under the tubing and adhere to skin.



Use adhesive strips for additional securement of the tubing sections and the hub of the catheter.