# Aquasonic 100® Ultrasound Transmission Gel

## Aquasonic CLEAR® Ultrasound Gel

### Single-Use Packettes

REF 01-20, 03-20

- EN** Instructions for use
- ES** Instrucciones de uso
- DE** Gebrauchsanweisung
- FR** Mode d'emploi
- IT** Istruzioni per l'uso
- PT** Instruções de utilização
- DA** Brugsanvisning
- FI** Käyttöohjeet
- NL** Gebruiksaanwijzing
- SV** Bruksanvisning
- EL** Οδηγίες Χρήσης
- HU** Használati útmutatás
- PO** Instrukcja stosowania
- HE** הוראות שימוש
- JA** 取扱説明書
- KO** 사용법
- ZH** 使用说明
- AR** رشادات الاستعمال

PARKER LABORATORIES, INC.

Parker Laboratories, Inc.  
286 Eldridge Road  
Fairfield, NJ 07004 USA  
(973) 276-9500  
[parkerlabs.com](http://parkerlabs.com)

Parker Laboratories BV  
Bedrijvenpark Twente 305  
7602 KL Almelo  
The Netherlands

EN

Aquasonic® 100 Ultrasound Transmission Gel  
Aquasonic CLEAR® Ultrasound Gel  
Single-Use Packettes

#### Instructions for Use:

#### Description

Aquasonic 100 Ultrasound Transmission Gel and Aquasonic CLEAR Ultrasound Gel Single-Use Packettes are non-sterile, water-soluble and non-staining gels intended for external use and natural body orifices only, and may be used with all commercially available ultrasound devices. For single use only.

#### Indications for Use

Aquasonic 100 Ultrasound Transmission Gel and Aquasonic CLEAR Ultrasound Gel Single-Use Packettes can be used on intact, unbroken skin and natural body orifices and on all patients in facilities where cross contamination is of concern, such as:

- Patients with immunodeficiency or on immunosuppressive therapy.
- Neonates or critically ill pediatric patients.
- Mucous membranes.
- Critically ill patients or patients in contact, airborne or droplet transmission based precautions.

Discard packette after use.

#### Warnings

- Do not use on patients with a known sensitivity to parabens.

*Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the EU Member State in which the user and/or patient is established.*

#### Precautions

- Discard product if package is damaged or evidence of contamination is present.
- Apply Aquasonic 100 Ultrasound Transmission Gel or Aquasonic CLEAR Ultrasound Gel to transducer surface, and not directly on to the patient.

#### Contraindications

Aquasonic 100 Ultrasound Transmission Gel and Aquasonic CLEAR Ultrasound Gel Single-Use Packettes should not be used in:

- Any invasive procedures in which a device is passed through the tissue, e.g. needle aspiration, needle localization, tissue biopsy, or other surgical procedures.
- Any procedure on or near broken skin, such as fresh surgical site or an open wound.
- Not to be used for procedures in the esophagus.

#### Storage Conditions

Store in a cool dry place. Store away from direct sunlight.

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