
Polysonic® Ultrasound Lotion

Polysonic® Ultrasound Lotion with Aloe Vera

REF 21-08, 21-28, 21-50

REF 20-08, 20-28, 20-50


EN	Instructions for use	KO	사용법
ES	Instrucciones de uso	ZH	使用说明
DE	Gebrauchsanweisung	AR	إرشادات الاستعمال
FR	Mode d'emploi	BG	Инструкции за употреба
IT	Istruzioni per l'uso	CZ	Návod k použití
PT	Instruções de utilização	ET	Kasutusjuhend
DA	Brugsanvisning	HR	Upute za upotrebu
FI	Käyttöohjeet	IS	Leiðbeiningar um notkun
NL	Gebruiksaanwijzing	LT	Naudojimo instrukcijos
SV	Bruksanvisning	LV	Lietošanas instrukcija
EL	Οδηγίες Χρήσης	NO	Bruksanvisning
HU	Használati útmutatás	RO	Instrucțiuni de utilizare
PO	Instrukcja stosowania	RU	Инструкция по применению
HE	הוראות שימוש	SK	Návod na použitie
JA	取扱説明書	SL	Navodila za uporabo

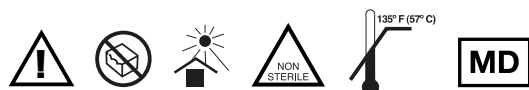
PARKER LABORATORIES, INC.

Parker Laboratories, Inc.
286 Eldridge Road
Fairfield, NJ 07004 USA
(973) 276-9500
parkerlabs.com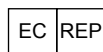

Parker Laboratories BV
Bedrijvenpark Twente 305
7602 KL Almelo
The Netherlands

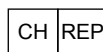MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau
Switzerland

EN

Polysonic® Ultrasound Lotion
Polysonic® Ultrasound Lotion with Aloe Vera

Instructions for Use:

Description

Polysonic Ultrasound Lotion and Polysonic Ultrasound Lotion with Aloe Vera are non-sterile, water-soluble and non-staining lotions intended for external use only and are used with all commercially available ultrasound devices.

Indications for Use

Polysonic Ultrasound Lotion and Polysonic Ultrasound Lotion with Aloe Vera can be used on intact, unbroken skin and on all patients in facilities where cross contamination is of minimal concern. Discard empty dispenser after use.

Refill containers and refillable dispensers (product numbers 20-28, 20-50, 21-28 and 21-50) are for use on low-risk patients and procedures in non-acute care facilities. Follow the enclosed directions for use or your facility's protocol, if it is different.

Contraindications

Polysonic Ultrasound Lotion and Polysonic Ultrasound Lotion with Aloe Vera is for external use only and should not be used in:

- Any invasive procedures in which a device is passed through the tissue, e.g. needle aspiration, needle localization, tissue biopsy, or other surgical procedures.
- Any procedure on or near broken skin, such as fresh surgical site or an open wound.
- Patients with an immunodeficiency or on immunosuppressive therapy.
- Any neonates or critically ill pediatric patients.
- Mucous membranes.
- Critically ill patients or patients in contact-, airborne- or droplet-transmission-based precautions.

Storage Conditions

Store in a cool dry place. Store away from direct sunlight.

Warnings and Precautions

Discard product if package is damaged or evidence of contamination is present. Do not use on patients with a known sensitivity to parabens.

To avoid contamination, tips of containers should not come into contact with patient, staff or instruments.

Apply Polysonic Ultrasound Lotion and Polysonic Ultrasound Lotion with Aloe Vera to transducer surface, and not directly on to the patient.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the EU Member State in which the user and/or patient is established.

Directions for Refilling Dispenser Bottles

Consult your facility policy before refilling from bulk container.

For refilling dispenser bottles, wipe the bottleneck and dispenser tip with isopropyl alcohol or a hospital-grade disinfectant, such as Protex® Disinfectant Wipes, just prior to filling.

Reusable containers should be washed in hot soapy water, rinsed and thoroughly dried before being refilled.

Isopropyl alcohol or a hospital-grade disinfectant, such as Protex Disinfectant Wipes, can be used to clean dispenser bottles.

Only refill product into corresponding labeled dispenser, and mark it with lot number.

Reusable bottles should be marked with the date filled and then discarded after one month.

Cracked or damaged bottles should be discarded immediately.

Reusable bottles should **not** be topped off. Contents should be consumed, or remaining gel should be disposed prior to refilling.

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