SCAN® Ultrasound Gel

REF 11-08, 11-28, 11-28s

- EN Instructions for use
- ES Instrucciones de uso
- DE Gebrauchsanweisung
- FR Mode d'emploi
- IT Istruzioni per l'uso
- PT Instruções de utilização
- DA Brugsanvisning
- FI Käyttöohjeet
- NL Gebruiksaanwijzing
- SV Bruksanvisning
- EL Οδηγίες Χρήσης
- HU Használati útmutatás
- PO Instrukcja stosowania
- HE הוראות שימוש
- JA 取扱説明書
- KO 사용법
- ZH 使用说明
- AR إرشادات الاستعمال

PARKER LABORATORIES, INC.

Parker Laboratories, Inc.
286 Eldridge Road
Fairfield, NJ 07004 USA
(973) 276-9500
parkerlabs.com

Parker Laboratories BV
Bedrijvenpark Twente 305
7602 KL Almelo
The Netherlands

EN

SCAN® Ultrasound Gel

Instructions for Use:

Description

SCAN Ultrasound Gel is a non-sterile, water-soluble and non-staining gel intended for external use only and used with all commercially available ultrasound devices.

Indications for Use

SCAN Ultrasound Gel can be used on intact, unbroken skin and on all patients in facilities where cross contamination is of minimal concern. Discard empty dispenser after use.

Refill containers and refillable dispensers (11-28 and 11-28s) for use on low-risk patients and procedures in non-acute care facilities. Follow the enclosed instructions for use or your facility's protocol, if it is different.

Contraindications

SCAN Ultrasound Gel, in multi-use packaging, is for external use only and should not be used in:

- Any invasive procedures in which a device is passed through the tissue, e.g. needle aspiration, needle localization, tissue biopsy, or other surgical procedures.
- Any procedure on or near broken skin, such as fresh surgical site or an open wound.
- Patients with an immunodeficiency or on immunosuppressive therapy.
- Any neonates or critically ill pediatric patients.
- Mucous membranes.
- Critically ill patients or patients in contact-, airborne- or droplet-transmission-based precautions.

Storage Conditions

Store in a cool dry place. Store away from direct sunlight.

Warnings and Precautions

Discard product if package is damaged or evidence of contamination is present.

Do not use on patients with a known sensitivity to parabens.

To avoid contamination, tips of containers should not come into contact with patient, staff or instruments.

Apply SCAN Ultrasound Gel to transducer surface, and not directly on to the patient.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the EU Member State in which the user and/or patient is established.

Directions for Refilling Dispenser Bottles

Consult your facility policy before refilling from bulk container.

For refilling dispenser bottles, wipe the bottleneck and dispenser tip with isopropyl alcohol or a hospital-grade disinfectant, such as Protex® ULTRA Disinfectant Wipes, just prior to filling.

Reusable containers should be washed in hot soapy water, rinsed and thoroughly dried before being refilled.

Isopropyl alcohol or a hospital-grade disinfectant, such as Protex ULTRA Disinfectant Wipes, can be used to clean dispenser bottles.

Only refill product into corresponding labeled dispenser, and mark it with lot number.

Reusable bottles should be marked with the date filled and then discarded after one month.

Cracked or damaged bottles should be discarded immediately.

Reusable bottles should **not** be topped off. Contents should be consumed, or remaining gel should be disposed prior to refilling.

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