# Signacreme® Electrode Cream

REF 17-05, 17-20

- EN Instructions for use
- ES Instrucciones de uso
- DE Gebrauchsanweisung
- FR Mode d'emploi
- IT Istruzioni per l'uso
- PT Instruções de utilização
- DA Brugsanvisning
- FI Käyttöohjeet
- NL Gebruiksaanwijzing
- SV Bruksanvisning
- EL Οδηγίες Χρήσης
- HU Használati útmutatás
- PO Instrukcja stosowania
- HE הוראות שימוש
- JA 取扱説明書
- KO 사용법
- ZH 使用说明
- AR إرشادات الاستعمال
- CZ Návod k použití

PARKER LABORATORIES, INC.

Parker Laboratories, Inc.  
286 Eldridge Road  
Fairfield, NJ 07004 USA  
(973) 276-9500  
parkerlabs.com

Parker Laboratories BV  
Bedrijvenpark Twente 305  
7602 KL Almelo  
The Netherlands

MDSS CH GmbH  
Laurenzenvorstadt 61  
5000 Aarau  
Switzerland

EN

Signacreme® Electrode Cream

## Instructions for Use:

### Description

Signacreme Electrode Cream is a non-sterile, electrically conductive cream. It is bacteriostatic.

### Indications for Use

Signacreme is recommended for use in electromedical procedures (except defibrillation), where a cosmetic quality cream is preferred.

### Contraindications

- Signacreme is not indicated for defibrillation procedures.

### Warnings

- Do not use on patients with a known history of sensitivity to parabens.

*Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the EU Member State in which the user and/or patient is established.*

### Precautions

- For external use only on intact, unbroken skin.
- Discard product if package is damaged or evidence of contamination is present.
- To avoid contamination of product, the dispenser bottle tip and refill dispenser pump tip should not come into contact with patient, staff or instruments.

### Directions for Use

Refer to the Instructions For Use provided by the manufacturer of the electromedical device being used for information on how Signacreme should be used with their product. Signacreme can be removed using a moistened soft cloth.

### Directions for Refilling Dispensers

Consult your facility policy before refilling from bulk dispenser container. For refilling dispensers, wipe the bottleneck and the dispenser cap with isopropyl alcohol or a hospital grade disinfectant, such as Protex® Disinfectant Wipes, prior to filling. Reusable containers and caps should be washed in hot soapy water, rinsed and thoroughly dried before being refilled. Only refill product into corresponding labeled dispenser bottle and mark it with lot number and expiry date. Cracked or damaged dispenser bottles should be discarded immediately. Reusable dispenser bottles should **not** be topped off. Contents should be consumed, or the remaining contents should be disposed of prior to refilling.

### Handling

No special handling required. Signacreme is non-flammable and non-corrosive.

### Storage

Protect product from direct sunlight.

### Disposal

Follow all federal, state, and local regulations for non-hazardous waste disposal.

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