Tensive® Conductive Adhesive Gel

REF 22-60

- EN Instructions for use
- ES Instrucciones de uso
- DE Gebrauchsanweisung
- FR Mode d'emploi
- IT Istruzioni per l'uso
- PT Instruções de utilização
- DA Brugsanvisning
- FI Käyttöohjeet
- NL Gebruiksaanwijzing
- SV Bruksanvisning
- EL Οδηγίες Χρήσης
- HU Használati útmutatás
- PO Instrukcja stosowania
- HE הוראות שימוש
- JA 取扱説明書
- KO 사용법
- ZH 使用说明
- AR إرشادات الاستعمال
- CZ Návod k použití

PARKER LABORATORIES, INC.

Parker Laboratories, Inc.
286 Eldridge Road
Fairfield, NJ 07004 USA
(973) 276-9500
parkerlabs.com

EC REP

Parker Laboratories BV
Bedrijvenpark Twente 305
7602 KL Almelo
The Netherlands

CH REP

MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau
Switzerland

EN

Tensive® Conductive Adhesive Gel

Instructions for Use:

Description

Tensive Conductive Adhesive Gel is a non-sterile, non-flammable, odorless conductive adhesive gel. It is bacteriostatic.

Indications for Use

Tensive Conductive Adhesive Gel is recommended for all electromedical procedures (except defibrillation), including TENS, EMG, EEG and other electromedical procedures.

Contraindications

- Tensive is not indicated for defibrillation procedures.

Warnings

- Do not use on patients with a known history of sensitivity to parabens.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the EU Member State in which the user and/or patient is established.

Precautions

- For external use only on intact, unbroken skin.
- Discard product if package is damaged or evidence of contamination is present.
- To avoid contamination of product, the tube container tips should not come into contact with patient, staff or instruments.

Directions for Use

Refer to the Instructions For Use provided by the manufacturer of the electromedical device being used for information on how Tensive should be used with their product. Typically, apply a thin, even film of Tensive Gel to the electrode surface by applying one drop on to the center of the electrode, then spread the gel over the electrode surface. Tensive can be removed using a moistened soft cloth.

Note: If an excess of gel is used, the electrode may not adhere properly to the skin.

Handling

No special handling required. Tensive is non-flammable and non-corrosive.

Storage

Protect product from direct sunlight.

Disposal

Follow all federal, state, and local regulations for non-hazardous waste disposal.

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