ECLIPSE® Probe Cover

REF 38-01, 38-03, 38-01-1

- EN** Instructions for use
- ES** Instrucciones de uso
- DE** Gebrauchsanweisung
- FR** Mode d'emploi
- IT** Istruzioni per l'uso
- PT** Instruções de utilização
- DA** Brugsanvisning
- FI** Käyttöohjeet
- NL** Gebruiksaanwijzing
- SV** Bruksanvisning
- EL** Οδηγίες Χρήσης
- HU** Használati útmutatás
- PO** Instrukcja stosowania
- HE** הוראות שימוש
- JA** 取扱説明書
- KO** 사용법
- ZH** 使用说明
- AR** إرشادات الاستعمال
- CZ** Návod k použití

PARKER LABORATORIES, INC.

Parker Laboratories, Inc.
286 Eldridge Road
Fairfield, NJ 07004 USA
(973) 276-9500
parkerlabs.com

Parker Laboratories BV
Bedrijvenpark Twente 305
7602 KL Almelo
The Netherlands

MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau
Switzerland

EN

ECLIPSE® Probe Cover

Instructions for Use:

Description

Eclipse Probe Cover (Product #38-01) is 2.37/1.73"(60 mm/44 mm) wide x 9.5" (241mm) long. Eclipse 3D Probe Cover (Product #38-03) is 3.25/1.73" (83 mm/44 mm) wide x 9.5" (241 mm) long. The probe covers are pre-filled inside with Aquasonic® 100 Ultrasound Transmission Gel. Eclipse Probe Covers are non-irritating, non-sensitizing and are not made with natural rubber latex.

Indications for Use

Eclipse Probe Covers are disposable probe covers for use with medical ultrasound probes/transducers.

Contraindications

- This product is non-sterile and not intended to be sterilized.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the EU Member State in which the user and/or patient is established.

Precautions

- Discard product if package is damaged or evidence of contamination is present.

Directions for Use

- 1) Hold the inner white tab while inserting a clean, dry probe/transducer head into the pre-gelled Eclipse Probe Cover. Gently slide probe until the head reaches the tip of the probe cover.
- 2) Hold the top printed blue tab and gently pull towards you to release the protective outer packaging.
- 3) Smooth the Eclipse Probe Cover over the probe/transducer head to eliminate air bubbles and cover the probe lens with gel. If air bubbles remain in lens area, gently smooth them with your thumb. The probe/transducer is now protected and ready for normal use.
- 4) The prepared probe should be used immediately to prevent the possibility of contamination.

Handling

No special handling required.

Storage

Protect product from direct sunlight.

Disposal

Follow all federal, state, and local regulations for non-hazardous waste disposal.

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