Redux® Creme Redux® Gel Redux® Paste

REF 66-04

REF 65-04

REF 67-05

- EN** Instructions for use
- ES** Instrucciones de uso
- DE** Gebrauchsanweisung
- FR** Mode d'emploi
- IT** Istruzioni per l'uso
- PT** Instruções de utilização
- DA** Brugsanvisning
- FI** Käyttöohjeet
- NL** Gebruiksaanwijzing
- SV** Bruksanvisning
- EL** Οδηγίες Χρήσης
- HU** Használati útmutatás
- PO** Instrukcja stosowania
- HE** הוראות שימוש
- JA** 取扱説明書
- KO** 사용법
- ZH** 使用说明
- AR** إرشادات الاستعمال
- CZ** Návod k použití

PARKER LABORATORIES, INC.

Parker Laboratories, Inc.
286 Eldridge Road
Fairfield, NJ 07004 USA
(973) 276-9500
parkerlabs.com

Parker Laboratories BV
Bedrijvenpark Twente 305
7602 KL Almelo
The Netherlands

MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau
Switzerland

EN

Redux®

Redux® Creme Description

Redux Creme is a non-sterile, electrically conductive creme. It is bacteriostatic.

Indications for Use

Redux Creme is recommended for routine ECG procedures.

Contraindications

- Redux Creme is not indicated for defibrillation procedures.

Precautions

- For external use only on intact, unbroken skin.
- Discard product if package is damaged or evidence of contamination is present.
- To avoid contamination of product, the tube container tips should not come into contact with patient, staff or instruments.

Redux® Gel

Description

Redux Gel is a non-sterile, non-abrasive, water based, electrically conductive gel.

Indications for Use

Redux Gel is recommended for routine ECG procedures and defibrillation.

Warnings

- Do not use on patients with a known history of sensitivity to parabens.

Precautions

- For external use only on intact, unbroken skin.
- Discard product if package is damaged or evidence of contamination is present.
- To avoid contamination of product, the tube container tips should not come into contact with patient, staff or instruments.

Redux® Paste

Description

Redux Paste is a bacteriostatic, non-sterile, water based, abrasive, electrically conductive paste.

Indications for Use

Redux Paste is recommended for short term monitoring, ECG, and defibrillation.

Precautions

- For external use only on intact, unbroken skin.
- Discard product if package is damaged or evidence of contamination is present.
- To avoid contamination of product, the tube container tips should not come into contact with patient, staff or instruments.

Redux® Paste, Creme, Gel

Directions for Use

Refer to the Instructions For Use provided by the manufacturer of the electromedical device being used for information on how Redux should be used with their product. Redux can be removed using a moistened soft cloth.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the EU Member State in which the user and/or patient is established.

Handling

No special handling required.

Storage

Protect product from direct sunlight.

Disposal

Follow all federal, state, and local regulations for non-hazardous waste disposal.

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