INSTRUCTIONS FOR USE

Description

Protex® Pro Disinfectant Wipes are premoistened, low-linting disinfecting wipes for the cleaning and disinfection of hard surfaces of non-invasive, non-porous, non-critical and non-lumened medical devices. Effective against bacteria, yeast and fungi.

Applications

The applications include – but are not limited to – skin surface ultrasound transducers and stations, IV poles, other hard non-porous, non-critical surfaces. Additional uses include monitors, keyboards, counter tops and probe holders.

Contraindications

This product is not intended to be used to disinfect invasive medical devices.

Warnings

- Contains polymeric biguanide hydrochloride
- May produce an allergic reaction
- Causes serious eye irritation

Precautions

- Wear protective gloves, protective clothing and eye protection.
- **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses if easy to do and continue rinsing. If eye irritation persists, get medical advice.
- Each wipe is for single use.
- Do not use on live electrical components
- Single-use wipes must not be reused. The medical device is intended to be used once on a single procedure and then discarded. The re-use of single-use medical devices can affect their safety, performance and effectiveness.
- Refer to the Safety Data Sheet of the product, as applicable.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the EU Member State in which the user and/or patient is established.

Directions for Use

- Wear gloves when handling disinfectants and medical devices.
- Before first use remove lid and foil seal. Feed the first wipe through the dispenser hole and pull.
- Replace the lid and close the cap.
- Wipe already contains active ingredients. Do not add water or other chemicals.
- Wipe equipment or surface as required.
- Ensure all areas are wet for the five-minute contact time. Leave surface to dry.

- Close the cap after each use to prevent remaining wipes from drying.
- Do not refill the canister or use it for other purposes.
- Dispose wipe to clinical waste – do not macerate, do not flush, do not reuse.
- For professional use only.
- Use disinfectants safely. Always read the label and product information before use. Always follow medical equipment manufacturers guidelines.

Contents

Active Ingredients:

Didecyl dimethyl ammonium chloride.....	0.23%
Alkyldimethyl benzyl ammonium chloride.....	0.13%
Polyhexamethylene biguanide.....	0.208%

Wipe: Polypropylene and viscose substrate

Biocidal Performance

Protex® Pro Disinfectant Wipes are effective against the following microorganisms of concern and also destroys DNA and RNA.

Candida albicans, *Trichophyton interdigitale*, *Acinetobacter baumannii*, *Campylobacter jejuni*, *Enterococcus faecalis*, *Enterococcus hirae*, *Escherichia coli*, *Klebsiella pneumoniae*, Methicillin-resistant *Staphylococcus aureus* (MRSA), Methicillin-resistant *Staphylococcus pseudintermedius* (MRSP), *Proteus vulgaris*, *Pseudomonas aeruginosa*, *Rhodococcus equi*, *Salmonella typhimurium*, *Staphylococcus aureus*, *Stenotrophomonas maltophilia*, *Streptococcus equi*, Vancomycin-resistant *Enterococci* (VRE) *Enterococcus faecium*

Handling

- Wear gloves when handling disinfectants and medical devices.

Storage

- Store in original canister in areas inaccessible to children. Keep securely closed. Nonrefillable canister. Do not reuse or refill the canister.
- Store the product within the specified temperature range.
- Once opened, use within six weeks. Discard if not used.

Disposal

- Dispose of wipes and canister in accordance with local policy and national regulations.

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Parker Laboratories, Inc.