
UltraDrape®

REF 34-15, 34-10

- EN Instructions for use
- ES Instrucciones de uso
- DE Gebrauchsanweisung
- FR Mode d'emploi
- IT Istruzioni per l'uso
- PT Instruções de utilização
- DA Brugsanvisning
- FI Käyttöohjeet
- NL Gebruiksaanwijzing
- SV Bruksanvisning
- EL Οδηγίες Χρήσης
- HU Használati útmutatás
- PO Instrukcja stosowania
- HE הוראות שימוש
- JA 取扱説明書
- KO 사용법
- ZH 使用说明
- AR إرشادات الاستعمال

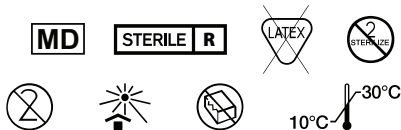PARKER LABORATORIES, INC.

Parker Laboratories, Inc.
286 Eldridge Road
Fairfield, NJ 07004 USA
(973) 276-9500
parkerlabs.com

Patent parkerlabs.com/ultradrape

Parker Laboratories BV
Bedrijvenpark Twente 305
7602 KL Almelo
The Netherlands

EN

UltraDrape® and UltraDrape® II

Instructions for Use:

Description

UltraDrape / UltraDrape II is a uniquely designed, dual-action barrier and securement Dressing for use during Ultrasound-Guided Peripheral Intravenous (UGPIV) procedures. The dressing is sterile, non-irritating, non-sensitizing and is not made with natural rubber latex.

UltraDrape is 5.8 in (14.72 cm) x 3.25 in (8.25 cm)

UltraDrape II is 4.68 in (11.9 cm) x 2.63 in (6.7 cm)

Ultrasound gel is applied to removable film layer to prevent contamination of injection site. Bifurcated, "stand-alone" design prevents gel from reaching IV site while enabling seamless catheter insertion and securement. Adhesive strips on tab #3 provide added securement of tubing and catheter hub surround.

Indications for Use

UltraDrape / UltraDrape II is a single-use, sterile, Ultrasound-Guided Peripheral Intravenous (UGPIV) barrier and securement device. It is intended to protect an injection site from ultrasound gel and to secure IV catheters to skin.

Any serious incident occurring in relation to device should be reported to the manufacturer and competent authority of EU Member State in which user and/or patient is established.

Precautions

- Discard product if package is damaged or evidence of contamination is present.
- Do not re-use.
- Do not re-sterilize.

UltraDrape / UltraDrape II Application Considerations

- Ensure skin is clean, free of soap residue and lotion, and allowed to dry thoroughly before applying dressing to prevent skin irritation and ensure good adhesion.
- Dressing may remain on skin for up to seven days. Replace dressing if it becomes damp, loosened, or visibly soiled.
- For additional information, refer to **Application Instructions**.

UltraDrape / UltraDrape II Removal Technique

- 1) Gently loosen top edge of dressing.
- 2) Place fingers on top of dressing to stabilize skin or catheter.
- 3) Remove dressing "low and slow" while stretching the dressing parallel to skin.

IMPORTANT: Removing dressing at an angle may pull at skin, increasing risk of skin injury.

Storage Conditions

Store between 10°C and 30°C, away from direct sunlight.

Disposal

Follow all federal, state, and local regulations for non-hazardous waste disposal.

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