Aquasonic 100® Ultrasound Transmission Gel Aquasonic CLEAR® Ultrasound Gel Single-Use Packettes

REF 01-20, 03-20

- EN** Instructions for use
- ES** Instrucciones de uso
- DE** Gebrauchsanweisung
- FR** Mode d'emploi
- IT** Istruzioni per l'uso
- PT** Instruções de utilização
- DA** Brugsanvisning
- FI** Käyttöohjeet
- NL** Gebruiksaanwijzing
- SV** Bruksanvisning
- EL** Οδηγίες Χρήσης
- HU** Használati útmutatás
- PO** Instrukcja stosowania
- HE** הוראות שימוש
- JA** 取扱説明書
- KO** 사용법
- ZH** 使用说明
- AR** إرشادات الاستعمال
- CZ** Návod k použití

PARKER LABORATORIES, INC.

Parker Laboratories, Inc.
286 Eldridge Road
Fairfield, NJ 07004 USA
(973) 276-9500
parkerlabs.com

Parker Laboratories BV
Bedrijvenpark Twente 305
7602 KL Almelo
The Netherlands

MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau
Switzerland

EN

Aquasonic® 100 Ultrasound Transmission Gel
Aquasonic CLEAR® Ultrasound Gel
Single-Use Packettes

Instructions for Use:

Description

Aquasonic 100 Ultrasound Transmission Gel and Aquasonic CLEAR Ultrasound Gel Single-Use Packettes are non-sterile, water-soluble and non-staining gels intended for external use and natural body orifices only, and may be used with all commercially available ultrasound devices. For single use only.

Indications for Use

Aquasonic 100 Ultrasound Transmission Gel and Aquasonic CLEAR Ultrasound Gel Single-Use Packettes can be used on intact, unbroken skin and on all patients in facilities where cross contamination is of concern, such as:

- Patients with immunodeficiency or on immunosuppressive therapy.
 - Neonates or critically ill pediatric patients.
 - Critically ill patients or patients in contact, airborne or droplet transmission based precautions.
- Discard packette after use.

Precautions

- Do not use on patients with a known sensitivity to propylene glycol, as this could cause an allergic reaction.
- Discard product if package is damaged or evidence of contamination is present.
- Apply Aquasonic 100 Ultrasound Transmission Gel or Aquasonic CLEAR Ultrasound Gel to transducer surface, and not directly on to the patient.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the EU Member State in which the user and/or patient is established.

Contraindications

- Aquasonic 100 Ultrasound Transmission Gel and Aquasonic CLEAR Ultrasound Gel Single-Use Packettes should not be used in:
- Any invasive procedures in which a device is passed through the tissue, e.g. needle aspiration, needle localization, tissue biopsy, or other surgical procedures.
 - Any procedure on or near broken skin, such as fresh surgical site or an open wound.
 - Not to be used for procedures in the esophagus.

Storage Conditions

Store in a cool dry place. Store away from direct sunlight.

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