



## Endocavity Probes

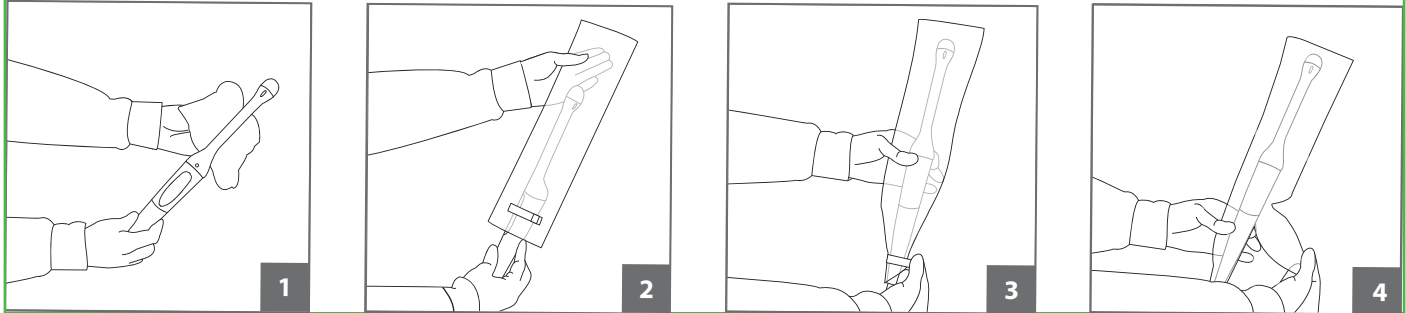

- 1 Clean & disinfect probe.
- 2 Insert probe into cover, covering strain relief.
- 3 Secure adhesive tab and store probe per facility protocol.
- 4 Remove cover by releasing adhesive tab and tear along perforation.

## General Purpose Probes

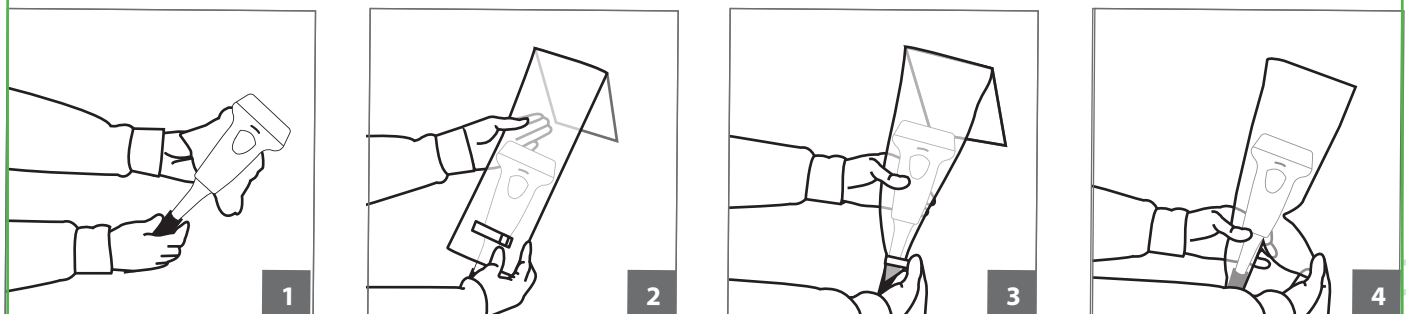

- 1 Clean & disinfect probe.
- 2 Insert probe into cover, covering strain relief.
- 3 Secure adhesive tab and store probe per facility protocol.
- 4 Remove cover by releasing adhesive tab and tear along perforation.



SCAN TO SHARE YOUR FEEDBACK