
AQUAFLEX® Ultrasound Gel Pad

REF 04-02, 04-15, 04-40, 04-42, 04-90, 04-92, 04-95

- EN** Instructions for use
- ES** Instrucciones de uso
- DE** Gebrauchsanweisung
- FR** Mode d'emploi
- IT** Istruzioni per l'uso
- PT** Instruções de utilização
- DA** Brugsanvisning
- FI** Käyttöohjeet
- NL** Gebruiksaanwijzing
- SV** Bruksanvisning
- EL** Οδηγίες Χρήσης
- HU** Használati útmutatás
- PO** Instrukcja stosowania
- HE** הוראות שימוש
- JA** 取扱説明書
- KO** 사용법
- ZH** 使用说明
- AR** إرشادات الاستعمال
- CZ** Návod k použití
- NO** Bruksanvisning

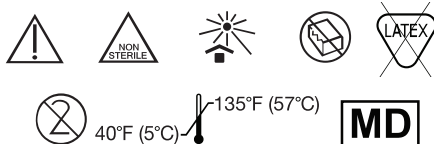PARKER LABORATORIES, INC.

Parker Laboratories, Inc.
286 Eldridge Road
Fairfield, NJ 07004 USA
(973) 276-9500
parkerlabs.com

Parker Laboratories BV
Bedrijvenpark Twente 305
7602 KL Almelo
The Netherlands

MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau
Switzerland

EN

AQUAFLEX® Ultrasound Gel Pad

Instructions for Use:

Description

AQUAFLEX is an aqueous, flexible, disposable ultrasound standoff.

AQUAFLEX cushions sensitive areas for comfortable scanning and/or therapeutic treatment of irregular surfaces such as bony prominences and small joints. The AQUAFLEX Ultrasound Gel Pad is solid, yet contours to irregular surfaces for accurate transmission.

Indications for Use

AQUAFLEX is recommended for diagnostic procedures performed on:

- Anterior gall bladder
- Breasts
- Carotid artery
- Neonatal heads
- Subcutaneous masses
- Testicles
- Thyroid
- Any procedure where near field visualization is required

Warnings

- Do not use on patients with a known history of skin allergies to parabens.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the EU Member State in which the user and/or patient is established.

Precautions

- For external use only on intact, unbroken skin.
- Discard product if package is damaged or evidence of contamination is present.

Directions for Use

Refer to the Instructions For Use provided by the manufacturer of the ultrasound device being used for information on how AQUAFLEX should be used with their product.

Handling

No special handling required.

Storage

Protect product from direct sunlight.

Disposal

Follow all federal, state, and local regulations for non-hazardous waste disposal.

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