# Signaspray® Electrode Solution and Skin Prep

REF 18-04, 18-25, 18-28

EN	Instructions for use	KO	사용법
ES	Instrucciones de uso	ZH	使用说明
DE	Gebrauchsanweisung	AR	إرشادات الاستعمال
FR	Mode d'emploi	BG	Инструкции за употреба
IT	Istruzioni per l'uso	CZ	Návod k použití
PT	Instruções de utilização	ET	Kasutusjuhend
DA	Brugsanvisning	HR	Upute za upotrebu
FI	Käyttöohjeet	IS	Leiðbeiningar um notkun
NL	Gebruiksaanwijzing	LT	Naudojimo instrukcijos
SV	Bruksanvisning	LV	Lietošanas instrukcija
EL	Οδηγίες Χρήσης	NO	Bruksanvisning
HU	Használati útmutatás	RO	Instrucțiuni de utilizare
PO	Instrukcja stosowania	RU	Инструкция по применению
HE	הוראות שימוש	SK	Návod na použitie
JA	取扱説明書	SL	Navodila za uporabo

PARKER LABORATORIES, INC.

 Parker Laboratories, Inc.  
286 Eldridge Road  
Fairfield, NJ 07004 USA  
(973) 276-9500  
parkerlabs.com

EC REP  
Parker Laboratories BV  
Bedrijvenpark Twente 305  
7602 KL Almelo  
The Netherlands

CH REP  
MDSS CH GmbH  
Laurenzenvorstadt 61  
5000 Aarau  
Switzerland

EN

Signaspray® Electrode Solution and Skin Prep

## Instructions for Use:

### Description

SignaSpray Electrode Solution and Skin Prep is a non-sterile, electrically conductive, non-gritty spray electrolyte. It is bacteriostatic.

### Indications for Use

SignaSpray is recommended for electromedical procedures (except defibrillation) or for use with muscle stimulation equipment.

### Contraindications

- SignaSpray is not indicated for defibrillation procedures.

*Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the EU Member State in which the user and/or patient is established.*

### Precautions

- For external use only on intact, unbroken skin.
- Discard product if package is damaged or evidence of contamination is present.
- To avoid contamination of product, spray nozzle or refill dispenser pump should not come into contact with the patient, staff or instruments.

### Directions for Use

Spray SignaSpray on the skin and apply the electrodes. Refer to the Instructions For Use provided by the manufacturer of the electromedical device being used for information on how SignaSpray should be used with their product. SignaSpray is water soluble and can be removed using a moistened soft cloth.

### Directions for Refilling Dispenser Bottles

Consult your facility policy before refilling from bulk dispenser container. For refilling spray bottles, wipe the bottleneck and dispenser tip with isopropyl alcohol or a hospital grade disinfectant, such as Protex® Disinfectant Wipes, prior to filling. Reusable containers should be washed in hot soapy water, rinsed and thoroughly dried before being refilled. Only refill product into corresponding labeled spray bottle, and mark it with lot number and expiry date. Cracked or damaged bottles should be discarded immediately. Reusable bottles should **not** be topped off. Contents should be consumed, or the remaining contents should be disposed of prior to refilling.

### Handling

No special handling required.

### Storage

Protect product from direct sunlight.

### Disposal

Follow all federal, state, and local regulations for non-hazardous waste disposal.

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